

SA Health Vaccine Distribution Centre

TagAlert® Instructions

The Polystyrene Esky received for an urgent vaccine delivery contains a TagAlert

The TagAlert is a digital single use monitor to be disposed of as you would for batteries.

The TagAlert temperature monitor will indicate if there has been a temperature deviation for the Immunisation Section clinical team to review.

Instructions for reading the TagAlert

Once you have removed the lid and ice bricks of the Esky you will see the TagAlert.

Step 1

Hold down the  button and check the TagAlert.

If the LCD displays an arrow to the tick ✓ no alarms have been triggered during transit. Remove vaccines and immediately place in vaccine fridge.

Step 2

If the LCD displays an arrow to the cross X and a number 1, 2, 3, or 4, then an alarm has been triggered. Contact the Immunisation Section clinical team on 1300 232 272. Remove the vaccines and immediately isolate the products within your vaccine fridge until instructions have been received from the Immunisation Section clinical team.

